TERUMO SURFLASH™ I.V. CATHETER

CATHETER INTRAVEINEUX / I.V. KATHETER / CATETER I.V. / CATETER INTRAVENOSO / CATETERE INTRAVENOSO / I.V. CATHETER / INFUSIONSKANYL / I.V. KATETER / I.V. KATETER (INTRA VENØS) / SUONENSISÄINEN KATETRI / ΕΝΔΟΦΛΕΒΙΟΣ ΚΑΘΗΤΗΡΑΣ / ΚΑΤΕΤΕΡ ДЛЯ ПУНКЦИИ ПЕРИФЕРИЧЕСКИХ ВЕН / KANIULA DO WLEWAŃ DOŻYLNICH / I.V. (INTRAVÉNÁS) KATÉTER / I.V. KATETR (INTRAVENÓZNÍ) / I.V. KATÉTER (INTRAVENÓZNY)

Instructions for Use / Mode d'emploi / Gebrauchsanweisung / Instrucciones de uso / Instruções para uso / Istruzioni per l'uso / Gebruiksaanwijzing / Bruksanvisning / Brugsanvisning / Bruksanvisning / Käyttöohjeet / Οδηγίες Χρήσης / Инструкции для пользования / Instrukcja Użycia / Használati utasítás / Návod na použití / Návod na použitie

ENGLISH

Current scientific evidence supports that medical devices manufactured from stainless steel alloys containing cobalt do not cause an increased risk of cancer or adverse reproductive effects.

CAUTION: Do not use if unit package is damaged. Do not attempt to re-insert a partially or completely withdrawn needle. Do not use scissors or other sharp objects near or at the insertion site. Due to the increased tendency for pediatric patients (in comparison to adult patients) to have a stronger reaction to physiologic stress that may be associated with certain procedures, appropriate precautionary measures should be taken when this device is used on a child. This product contains metal material. Careful attention should be paid when using the product where Magnetic Resonance Imaging (MRI) is anticipated. Catheter should be changed according to CDC guidelines or facility policy. Observe Universal Precautions on all patients. Reuse increases risk of infection.

1. Perform venipuncture. 2. Confirm that the needle and catheter have been properly inserted into the vein through blood flashback in the flashback chamber and the catheter itself. 3. Holding the flashback chamber stationary, only advance the catheter forward into the vein. 4. Holding the colored hub stationary, withdraw the needle and discard safely. 5. Connect an administration set and secure. 6. Dispose of safely according to local regulations to avoid risk of infection.

Report serious incident to the national authority.

Radio-opaque. Non pyrogenic. Do not store at extreme temperatures and humidity.

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© TERUMO CORPORATION Sep. 2023 23G05