optilube.™

sterile lubricating jelly

CE 2797


STERILE R

OL-FU-Ju20-V12.0

EC REP

MT Promedt Consulting GmbH
Altenhofstrasse 80
66386 St. Ingbert Germany

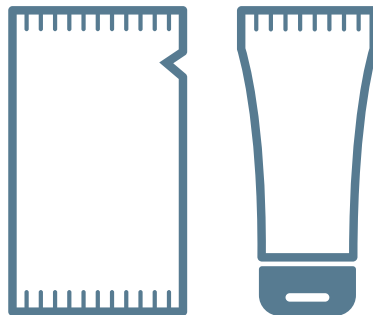Instructions for use

optimummedical®

Optimum Medical Solutions Ltd

Tennant Hall, Blenheim Grove,
Leeds, LS2 9ET, United Kingdom

T: +44(0) 845 643 5479

enquiries@optimummedical.co.uk

www.optimummedical.co.uk

BRAND: OptiLube

COMPOSITION:

Sterile, transparent, water-soluble jelly.

- Purified water • Polyethylene Glycol (PEG) • Glycerine • Sodium hydroxide
- 4-Hydroxybenzoic acid esters • pH range 5.0 – 7.0

INDICATIONS FOR USE:

OptiLube is a water soluble, sterile lubricating jelly used to aid the smooth insertion of medical devices in procedures including:

- Insertion of catheters • Endoscopy • Gynaecological examinations
- Rectal examinations • Cystoscopy

OptiLube is harmless to human tissue.

INSTRUCTIONS FOR USE:

SACHETS: Remove sachet from box, tear sachet at one end and apply. **OptiLube** 20g sachet is double-wrapped. Please use an aseptic technique to open and follow the above instructions. Single use only.

TUBES: Remove cap and peel foil from end of tube. Apply **OptiLube** sterile lubricating jelly to instrument or skin. Close after use. Once opened, use within 3 months. Sterile until opened.

REACH: Snap nozzle tip to open. Apply as necessary. Dispose of after use. Single use only.

CONTRAINDICATIONS: Do not use if you have a known sensitivity or allergy to any of the ingredients.

SIDE EFFECTS: There are no known side effects, however, if irritation occurs discontinue using and consult a doctor.

WARNINGS: Do not use if product is damaged or seal is broken. **OptiLube** must only be used under the direction of health professionals in accordance with local policies and procedures, including infection control guidelines. Ensure compatibility with medical devices being used. Apply the required amount of lubrication necessary for the procedure. Not for use in the eye.

STORAGE: Keep dry and out of direct sunlight. Reapply cap in between use. Unopened packs have a 5 year shelf life. Do not use after expiry date.

PACKAGING: Do not use if packaging is damaged or spoiled. Dispose of according to local policies and procedures.

PRESENTATION:

Tubes: 5g, 10g, 42g, 56.5g, 82g, 113g
Sachets: 2.7g, 3g, 5g, 20g
Reach: 5g, 15g

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COMPOSITION :

Sterile, transparent, water-soluble jelly.

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